110 mm

**Wirksamkeit<sup>1)</sup> – Wischdesinfektion / Efficacy<sup>1)</sup> – Wipe disinfection / Účinnost<sup>1)</sup> – Dezinfekce otřením / Účinnost<sup>1)</sup> – Razkuževanje z brisanjem / Veiksmīgums<sup>1)</sup> – Valomoji dezinfekcija / Verkan<sup>1)</sup> – Desinficērande avtorkning:**

Bakterizid/Levurozid / Bactericidal/Yeasticidal / Baktericidní/ Fungicidní na kvasinky / Baktericidno/Glive kvasovke / Baktericid/ Levurocid / Baktericid/Levurocid (EN 13727 / EN 13624 / EN 16615)	1 min
Mykobakterizid / Mycobactericidal / Mykobaktericidní / Mikobaktericidno / Mikobaktericidinis / Mykobaktericid	3 min
Begrenzt viruzid / Virucidal activity against enveloped viruses / Virucidní na obalené viry / Virucidno proti virusom z ovojnico / Virucidinis, prieš apvalkalęj turinčius virusus / Virucid mot kapslade virus (EN 14476)	1 min
Begrenzt viruzid PLUS / Limited spectrum virucidal activity / Omezené spektrum virucidní účinnosti / Omejeno virucidno delovanje / Ribotas virusidinis veikimas / Begränsat spektrum för virucidaktivitet (EN 14476)	30 min
Bakterizid/Levurozid / Bactericidal/Yeasticidal / Baktericidní/Fungicidní na kvasinky / Baktericidno/Glive kvasovke / Baktericid/Levurocid / Baktericid/Levurocid (EN 1276 / EN 1650 / EN 13697, 20°C)	1 min

1) Hohe Belastung / Dirty conditions / Spīnavē podmīnky / Umazane pogoji / Nešvarios sąlygos / Hög belastning

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**Medical device:** Bacillol 30 Sensitive Foam is a low-alcohol based, ready-to-use disinfectant for surfaces of non-invasive medical devices especially made of sensitive material. **Intended user:** Professional, trained users in a clinical context. **Biocidal product:** Ready-to-use liquid. For cleaning disinfection of washable surfaces in the medical and food processing area (PT2, PT4). **Intended user:** Professional, trained users. **Medical device + Biocidal product:** **Instructions for use:** Spray Bacillol 30 Sensitive Foam directly onto the surface, ensuring adequate coverage, then spread using a wipe. **Note:** When switching from one product to another, carry out an intermediate cleaning. It needs to be made sure that no disinfectant gets inside electronic devices. Store at room temperature in the original container. Keep tightly closed. After opening use within 12 months. **Wirkstoffe:** Medical Device. **Active substances:** Ethanol: 140 mg/g; Propan-2-ol: 100 mg/g; Propan-1-ol: 60 mg/g; Amines, N-C10-C16-alkyltrimethylendiol-, reaction products with chloroacetic acid: 2 mg/g. **Labeling:** Flammable liquid and vapour. Causes serious eye irritation. Avoid contact with eyes.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Should not be released into the environment. Dispose of contents/container in accordance with local and national regulations.

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☞ **Medical device:** Bacillol 30 Sensitive Foam is a low-alcohol based, ready-to-use disinfectant for surfaces of non-invasive medical devices especially made of sensitive material. **Intended user:** Professional, trained users in a clinical context. **Biocidal product:** Ready-to-use liquid. For cleaning disinfection of washable surfaces in the medical and food processing area (PT2, PT4). **Intended user:** Professional, trained users. **Medical device + Biocidal product: Instructions for use:** Spray Bacillol 30 Sensitive Foam directly onto the surface, ensuring adequate coverage, then spread using a wipe. **Note:** When switching from one product to another, carry out an intermediate cleaning. It needs to be made sure that no disinfectant gets inside electronic devices. Store at room temperature in the original container. Keep tightly closed. After opening use within 12 months. **Medizinprodukt.** **Active substances:** Ethanol: 140 g/kg; Propan-2-ol: 100 g/kg; Propan-1-ol: 60 g/kg; Amines, N-C12-14(even numbered)-alkyltrimethylenedi-, reaction products with chloroacetic acid: 2 g/kg. **Labelling:** Flammable liquid and vapour. Causes serious eye irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Dispose of contents/container in accordance with local and national regulations.

Use biocides safely and sustainably. It is illegal to use this product for uses or in a manner other than prescribed on this label. Poisons Information: For information or to report a poisoning incident contact The National Poisons Information Centre (01 8092166). **PCS 101226. Notification holder:** BODE Chemie GmbH, Melanchthonstraße 27, 22525 Hamburg, Tel. +49 40 54 006-0.

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**BODE Chemie GmbH, GERMANY.**