schülke →

chirosan® plus

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EN **Concentrated powder disinfectant based on generated peracetic acid for single-phase disinfection, second and high level of disinfection of medical tools, operating, anesthetic and dental instrumentarium and endoscopes. Suitable also for disinfection of surfaces of medical devices.**

Instruction for use: Mix the requisite amount of the preparation thoroughly in water at a temperature of 20 to 25 °C. A 1% disinfectant solution is prepared by dissolving 10 g of chirosan® plus in 990 g of water. After a fifteen-minute activation and stirring once again the preparation is ready for use. The used instruments and resources are submerged fully in the prepared disinfectant solution.

After the prescribed period of exposure the instruments are rinsed in demineralised potable or sterile water and prepared for sterilisation. Disinfection of hydrotherapy baths is done with 2% chirosan® plus solution, after exposure time the surface is thoroughly rinsed with hot water. The preparation is also suitable for use in ultrasonic cleaners. It is not suitable for use on instruments and medical devices made of brass, copper, rubber or aluminium, or on nickel or chrome plated instruments with a mechanically disrupted surface. The working solution is prepared fresh every day. **Contains:** Sodium percarbonate, (1-hydroxyethyliden) bisphosphonic acid, sodium salt. **Notice: Only for professional use. Danger! H302** Harmful if swallowed. **H318** Causes serious eye damage. **P260** Do not breathe dust. **P280** Wear protective rubber gloves / protective clothing / eye protection / face protection. **P305 + P351 + P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P314** Get medical advice / attention if you feel unwell. **First aid:** Any serious incidents related to the product are to be reported to the manufacturer and competent authority. **Inhalation:** remove the source of exposition. Bring the victim to the fresh air; keep at rest (avoid even walking) if necessary, seek medical advice. **Skin contact:** Remove affected clothing. Wash thoroughly with water and soap. **Ingestion:** Rinse mouth with potable water and leave victim to drink 0,5 l of water. Do not induce vomiting, seek medical aid. **Storage:** Store the product in tightly closed original containers. Keep away from food, drink, fresh water, feed and flammable materials. Protect against direct sunlight. **Storage temperature:** from -10 to +25 °C. **Instruction for the safe disposal of product and packaging:** Apply the remaining preparation in accordance with instructions of use, or submit to authorised companies for disposal. Empty packaging should be thoroughly washed out and can then be recycled.

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