**CS** **GAMAPLUS**

**Katalogové č.:**  
V606301-ND  
V606301-01ND  
V606303-ND  
V606304-ND  
V606304-01ND

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EN **GAMAPLUS**

<b>Catalogue No.:</b> V606301-ND V606301-01ND V606303-ND V606304-ND V606304-01ND	<b>Medical device variants:</b> GAMAPLUS 1,8 x 450 LL GAMAPLUS 1,8 x 450 LLK GAMAPLUS 1,8 x 1500 LL GAMAPLUS 1,8 x 1800 LL GAMAPLUS 1,8 x 1800 LLK	<b>Catalogue No.:</b> V606306-ND V606307-ND V606321-ND V606342-ND	<b>Medical device variants:</b> GAMAPLUS 1,8 x 450 UNIV GAMAPLUS 1,8 x 1800 UNIV GAMAPLUS 3,0 x 150 LL GAMAPLUS 3,0 x 300 LL	<b>Catalogue No.:</b> V606345-ND V606347-ND V606349-ND V606371-ND	<b>Medical device variants:</b> GAMAPLUS POCARD GAMAPLUS 3,0 x 2500 LL GAMAPLUS 3,0 x 3500 LL GAMAPLUS 3,0 x 150 YLL
<b>Product type:</b>	medical device				
<b>Design:</b>	sterile, ethylene oxide sterilization				
<b>Target patient:</b>	population of pediatric or adult patients				
<b>Designated user:</b>	medical personnel with appropriate education, knowledge and practical experience in medical devices.				
<b>Materials used:</b>	PVC, PE				
<b>Compatibility:</b>	With medical devices equipped with standard Luer connectors in conformity with standards ČSN EN ISO 80369-7. GAMAPLUS is designed to transfer liquid or i.v. administration or transfer of blood. (In)compatibility of the drugs with the materials used, check in the summary about products (SPC) see <a href="http://www.sukl.cz">www.sukl.cz</a> or <a href="http://www.olecich.cz">www.olecich.cz</a>				
<b>Contraindication:</b>	Do not use on any patient with a known allergic reaction to any of the product components.				
<b>Adverse effects:</b>	No adverse effects are known if the set is used in accordance with the instructions specified in the Warning and Use sections.				
<b>Warning:</b>	Do not use if the sterile packing is opened or damaged, or when the protective covers dropped off. Do not use after the expiry date. Repeated use or repeated sterilization creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability, may lead to injury, illness or death of the patient.				
<b>Storage:</b>	The products should be stored in dry, ventilated, dust-free and dark rooms without direct sunlight to avoid their damage and deformation. Products must be stored in the packaging as delivered by the producer. Products must be stored at temperatures from +5°C to +25°C and the relative air humidity is max. 75%.				

**Intended use**

GAMAPLUS hose is intended for interconnection of one or two infusion or transfusion kits with the intravenous needle.

**Instruction for use**

- Take GAMAPLUS out of the packing under aseptic conditions.
- Remove the covers.
- Attach the negative (female) cone to the positive (male) cone of the set and fill the hose with liquid.
- Connect the positive cone to the intravenous needle or a catheter.

**Procedure for safe disposal of medical devices after use:**

Put the medical device in a blue PE bag intended for hazardous - infectious waste. After filling, seal the labeled bag (waste code, place of origin, date and signature of the responsible person) and transport it safely to the incinerator = decontamination device. Removal of the dangerous property of infectivity means proper decontamination. Waste sorting is carried out according to the operating rules of the medical facility.

**NOTE TO USERS:** Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.

**Used symbols on packaging**

	Catalogue Number		Keep away from Rain		Batch Number		Manufacturer
	Use by / Expiry date		Temperature Limitation		Read instructions for use		Sterilised by Ethylene Oxide
	Pyrogen free		Keep away from sunlight		Latex free product		Does not contain DEHP phthalate
	Do not Resterilize		For single use only		Do not use if unit package is damaged		Pressure
	Volume of tubing		Medical Device		Date of Manufacture		

The instruction for use is also available in electronic form at [www.gama.cz](http://www.gama.cz). Free copies of other copies are available on request via e-mail: [sales@gama.cz](mailto:sales@gama.cz)

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