Medix

Jednorazowe podkłady medyczne
Disposable medical couch roll/sheets
Einweg-Hygieneschutzunterlagen
Jednorázové lékařské/lekárske podložky

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WEBER & WEBER Sp. z o.o.
Puńców, ul. Cieszyńska 229, 43-400 Cieszyń
tel. +48 33 852 91 00, fax +48 33 852 92 00
www.webermed.com

Pro[®]

Paper+Foil

PF

WEBER & WEBER
medical production

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N: NONWOVEN VLIESTOFF.

EN**MEDIXPRO®
– DISPOSABLE MEDICAL COUCH ROLL/SHEETS****DEVICE DESCRIPTION:**

MedixPro® disposable medical couch roll/sheets are medical devices, intended to insulate the patient from potentially contaminated surfaces of medical couches, gynaecological armchairs, paediatric tables, examination tables for infants, procedure tables, tables for healing massages, physiotherapeutic tables, etc.

The use of disposable medical sheets improves the hygienic conditions of performed procedures and ensures reduction of the probability of peri-operative infections. The disposable medical sheets, with their additional film layer, may also be used at medium risk of contamination with fluids.

INTENDED USE:

MedixPro® disposable medical couch roll/sheets are intended to improve hygienic conditions during medical procedures, as well as to protect patient, furniture and medical equipment against procedure-derived contaminations. The disposable medical sheets may additionally be used to protect patient, furniture and medical equipment against disease-causing peri-operative infections.

USE INSTRUCTIONS

- Unwind the roll.
- Tear off at perforation line.
- Spread the unfolded pad at indicated site.
- Remove to utilisation after single use.

This medical device is intended for professional use. After single use, drop the pad into a foil bag and proceed further in line with applicable regulations on medical waste. MedixPro® disposable medical couch roll/sheets are non-sterile, non-invasive, disposable medical devices of limited exposure (up to 24 hours).

CONTRAINDICATIONS FOR USE

- Do not use the medical device in persons sensitized to polyethylene.
- Do not apply the sheet directly onto injured skin, open, unprotected wounds or ulcerations.

WARNINGS:

- This medical device is intended for single use. Do not reuse the medical device.
- During medical procedures with usage of the product you need to follow basic hygienic rules. The usage of the product do not exempt from decontamination (cleaning, washing and disinfection) working surface where the product is used.
- In order to protect infection, the device after single use, should be treated as contaminated or potentially contaminated waste, placed in special, intended containers and proceed in line with applicable regulations on medical waste.
- Store the medical device at dry areas. Do not expose the device onto direct sun light. Protect the device against permanent/considerable moisturing or becoming wet.
- Do not use the device if its packaging has previously been opened or damaged.
- The expiry period of the medical device is 3 years from its production date, specified on the label. Do not use after the expiry date.

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