

Chirana

Recommended Use Insulin and Tuberculin Syringe for Single Use



STERILE EO


- Description:** Insulin syringes for single use. Tuberculin syringes for single use. Sterile, Non-toxic, Non-pyrogenic.
- Intended use for:** Insulin syringes are intended for the administration of 1 ml; 0,5ml or 0,3 ml, that is 100, 50 or 30 units of U 100 insulin, or 1ml of 40 units of U 40 Insulin. Tuberculin syringes are intended for the administration of vaccines immediately after filling. They are not intended to store vaccines for prolonged period of time.
- Contraindication:** Medical device should not be used other than as indicated in intended use. Medical device should not be used for administration of blood.



1. Before usage confirm the sterility validity, packing completeness, the LOT No. and expiry date;
2. The products should be used immediately after unit package opened, discard after use;
3. It is not allowed for use if unit package damaged and validity expired;
4. To be used by qualified doctor or paramedic only;
5. Only for single use. Discard after single use;
6. This device does not contain phthalates DEHP;
7. Re-sterilization or re-use could cause infection or cross-interactions.



1. Before usage, open the unit package;
2. Examine individual package for integrity of packaging prior to use. If packaging has been torn, do not use;
3. Any used syringe is considered contaminated and should be disposed of in an approved container immediately after use;
4. Dispose of safely after single use to avoid risk of infection.

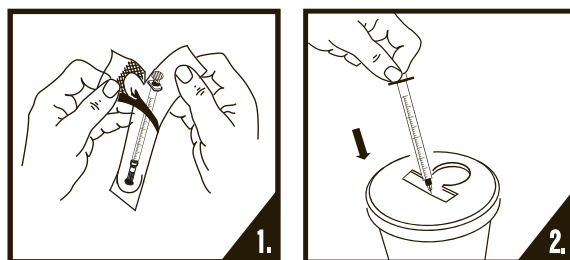

Attention



See instruction for use

REF:

Product reference/Art. No.



For single use only



Do not Resterilize

LOT

Batch Number



Date of Manufacturing



Use by / Expiry date

STERILE EO

Sterilised by Ethylene Oxide



Manufactured by



Pyrogen free



DEHP Free product



PVC Free product



LATEX Free product



Do not use if unit package is damaged.



Keep away from sunlight.



Temperature Limitation.



Keep away from Rain.